



Working Standard
Anti-Echinococcus Serum, Human
NIBSC code: ECHS
Instructions for use
(Version 8.0, Dated 18/05/2015)

This material is not for in vitro diagnostic use.

1. INTENDED USE

The reagent was examined by six laboratories and was reactive in several serological assays for Echinococcus. Cross reactivity was observed with Taenia saginata and Cysticercus fasciolaris. No cross-reactivity was observed with Fasciola hepatica, Ascaris suum and Diptelonema vitae. Please note that ECHS was considered no longer fit for use in modern diagnostic techniques by ECBS and delisted as a WHO IS in 2013.

2. CAUTION

This preparation is not for administration to humans or animals in the human food chain.

The preparation contains material of human origin, and either the final product or the source materials, from which it is derived, have been tested and found negative for HBsAg, anti-HIV and HCV RNA.

As with all materials of biological origin, this preparation should be regarded as potentially hazardous to health. It should be used and discarded according to your own laboratory's safety procedures. Such safety procedures should include the wearing of protective gloves and avoiding the generation of aerosols. Care should be exercised in opening ampoules or vials, to avoid cuts.

3. UNITAGE

One ampoule contains 87 mg of freeze dried serum. Assigned content of vial valid at time of manufacture – no information on long term stability.

4. CONTENTS

Country of origin of biological material: Lebanon and New Zealand.
The standard contains freeze-dried serum, which should be reconstituted with 1 ml sterile water. No preservatives have been added.

5. STORAGE

Please store reconstituted material at 4°C.

Please note: because of the inherent stability of lyophilized material, NIBSC may ship these materials at ambient temperature.

6. DIRECTIONS FOR OPENING

DIN ampoules have an 'easy-open' coloured stress point, where the narrow ampoule stem joins the wider ampoule body. Various types of ampoule breaker are available commercially. To open the ampoule, tap the ampoule gently to collect material at the bottom (labelled) end and follow manufacturers instructions provided with the ampoule breaker.

7. USE OF MATERIAL

No attempt should be made to weigh out any portion of the freeze-dried material prior to reconstitution

The standard should be reconstituted with 1 ml sterile water. An appropriate dilution of the standard can be used in bioassays or immunoassays for the laboratory diagnosis of Echinococcus infection.

8. STABILITY

NIBSC follows the policy of WHO with respect to its reference materials.

9. REFERENCES

WHO Technical report series 1976; 594:17.

10. ACKNOWLEDGEMENTS

none

11. FURTHER INFORMATION

Further information can be obtained as follows;

This material: enquiries@nibsc.org

WHO Biological Standards:

<http://www.who.int/biologicals/en/>

JCTLM Higher order reference materials:

<http://www.bipm.org/en/committees/jc/jctlm/>

Derivation of International Units:

http://www.nibsc.org/standardisation/international_standards.aspx

Ordering standards from NIBSC:

<http://www.nibsc.org/products/ordering.aspx>

NIBSC Terms & Conditions:

http://www.nibsc.org/terms_and_conditions.aspx

12. CUSTOMER FEEDBACK

Customers are encouraged to provide feedback on the suitability or use of the material provided or other aspects of our service. Please send any comments to enquiries@nibsc.org

13. CITATION

In all publications, including data sheets, in which this material is referenced, it is important that the preparation's title, its status, the NIBSC code number, and the name and address of NIBSC are cited and cited correctly.



14. MATERIAL SAFETY SHEET

Physical and Chemical properties	
Classification in accordance with Directive 2000/54/EC, Regulation (EC) No 1272/2008: Not applicable or not c Physical appearance: Off white powder	Corrosive: No
Stable: Yes	Oxidising: No
Hygroscopic: Yes	Irritant: No
Flammable: No	Handling: See caution, Section 2
Other (specify): Contains material of human origin	
Toxicological properties	
Effects of inhalation:	Not established, avoid inhalation
Effects of ingestion:	Not established, avoid ingestion
Effects of skin absorption:	Not established, avoid contact with skin
Suggested First Aid	
Inhalation:	Seek medical advice
Ingestion:	Seek medical advice
Contact with eyes:	Wash with copious amounts of water. Seek medical advice
Contact with skin:	Wash thoroughly with water.
Action on Spillage and Method of Disposal	
Spillage of ampoule contents should be taken up with absorbent material wetted with an appropriate disinfectant. Rinse area with an appropriate disinfectant followed by water. Absorbent materials used to treat spillage should be treated as biological waste.	

15. LIABILITY AND LOSS

In the event that this document is translated into another language, the English language version shall prevail in the event of any inconsistencies between the documents.

Unless expressly stated otherwise by NIBSC, NIBSC's Standard Terms and Conditions for the Supply of Materials (available at http://www.nibsc.org/About_Us/Terms_and_Conditions.aspx or upon request by the Recipient) ("Conditions") apply to the exclusion of all other terms and are hereby incorporated into this document by reference. The Recipient's attention is drawn in particular to the provisions of clause 11 of the Conditions.

16. INFORMATION FOR CUSTOMS USE ONLY

Country of origin for customs purposes*: United Kingdom * Defined as the country where the goods have been produced and/or sufficiently processed to be classed as originating from the country of supply, for example a change of state such as freeze-drying.
Net weight: 0.09 g
Toxicity Statement: Non-toxic
Veterinary certificate or other statement if applicable. Attached: No

17. CERTIFICATE OF ANALYSIS

NIBSC does not provide a Certificate of Analysis for WHO Biological Reference Materials because they are internationally recognised primary reference materials fully described in the instructions for use. The reference materials are established according to the WHO Recommendations for the preparation, characterization and establishment of international and other biological reference standards http://www.who.int/bloodproducts/publications/TRS932Annex2_Inter_biol_efstandardsrev2004.pdf (revised 2004). They are officially endorsed by the WHO Expert Committee on Biological Standardization (ECBS) based on the report of the international collaborative study which established their suitability for the intended use.